



A fresher face...
without
surgery.

pellevé[®]
RF WRINKLE REDUCTION



Introducing Pellevé®

Pellevé is a revolutionary innovation for smoother, tighter skin... FDA-cleared to treat facial wrinkles without surgery.

With Pellevé, you can look and feel refreshed, naturally.



Before

After 4 Months Post 2 Treatments

Courtesy of Stuart Bankover, MD



Before

After 2 Treatments

Courtesy of John Burke, MD

Put a fresh face forward

Pellevé® precisely heats the deep layers of your skin using advanced radiofrequency energy without damaging the epidermis, or top layer. The heat causes the existing collagen in your skin to contract and tighten. Also after the treatment, your skin will start to produce new collagen. The result is an improvement in skin quality, firmness and tightness you can see and feel.

Pellevé can be used to treat the skin on the face, including:

- Sagging skin under the chin
- Nasolabial folds
- Wrinkles around the mouth
- Lines or folds between the eyebrows
- Lines on the forehead
- Loose skin above or below the eye
- Crow's feet



Before

After 30 Days Post 2 Treatments

Courtesy of Igor Jeremic, MD

Results you can see in just a few treatments

Although many people start to see results immediately after the first treatment, most patients begin with a package of 3-4 treatments spaced about 30 days apart. Your doctor will recommend the right number of treatments for you.



Before

After 60 Days Post 3 Treatments

Courtesy of Red Alinsod, MD



Before

After 60 Days Post 5 Treatments

Courtesy of Igor Jeremic, MD

Frequently asked questions

Q: How is Pellevé® different from other skin tightening treatments?

A: Pellevé has been proven to provide safe and effective skin tightening in medical studies. Because Pellevé is a non-invasive procedure, meaning no surgery is required, there are minimal side effects and discomfort. Unlike other systems, no anesthesia or skin cooling is needed during your Pellevé procedure, so you can return to home or work immediately following treatment.

Q: What should I expect during the procedure?

A: You will feel a deep warming sensation each time the Pellevé handpiece touches the area being treated. Because no anesthetic is required, you'll be able to describe the depth of the heat to your doctor, which will help to ensure the best possible results.

Q: How long does a Pellevé procedure take?

A: Your Pellevé procedure will typically take about 45 to 60 minutes for a full facial treatment. Because Pellevé doesn't require anesthesia or downtime, it fits easily into your busy schedule.



www.Facebook.com/EllmanInternational

www.twitter.com/PelleveSkin

Q: What should I expect afterwards?

A: You should expect to be able to return to your daily activities with no downtime at all. Unlike lasers, there is no restriction from being in the sun and you can also apply makeup immediately after the procedure. Some patients experience mild swelling and redness which typically goes away within 2 to 24 hours.

Feel refreshed.

For the best results, it is important to choose a practice that is experienced in administering Pellevé®.

Talk to the healthcare professional that takes care of your skin, or visit www.pelleve.com to locate a doctor near you.



Before

After 60 Days Post 5 Treatments

Courtesy of Igor Jeremic, MD



Courtesy of Reynaldo Javate, MD

Before

After 30 Days Post 1 Treatment

To find more information on Pellevé® visit www.pelleve.com

Pellevé is indicated for the non-ablative treatment of mild to moderate facial wrinkles for all skin colors. Possible side effects of Pellevé are usually treatment-site related and include mild discomfort during the procedure within the treatment area. Mild swelling (edema) and redness (erythema) may occur during or after a Pellevé skin treatment. Talk to your doctor to see if Pellevé is right for you.

pellevé®
RF WRINKLE REDUCTION

ellman®
400 Karin Lane, Hicksville, N.Y. 11801 U.S.A.
(800) 835-5355 • (516) 594-3333 • Fax: (516) 279-4483
www.pelleve.com • www.ellman.com
© 2014 Ellman All Rights Reserved.

ellman®
A CYNODURE® Company